


HOW TO TAKE A MILK SAMPLE




1. RECORD

Record cow number, quarter and date on the pottle.
Use clean gloves

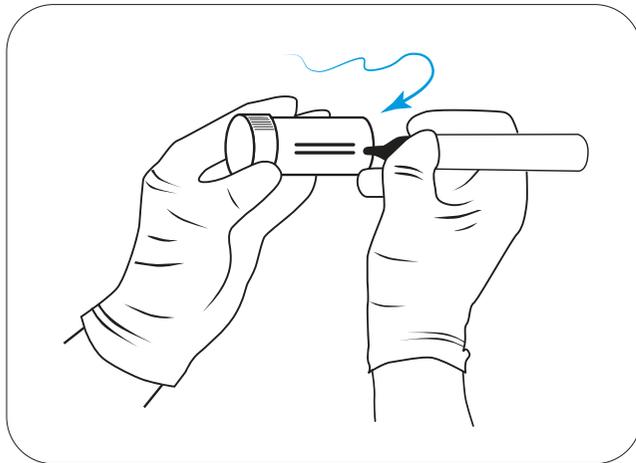


2. CLEAN TEAT

Scrub each teat end with a teat wipe or cotton wool ball soaked
in 17% meths. Use new wipe / cotton ball until clean

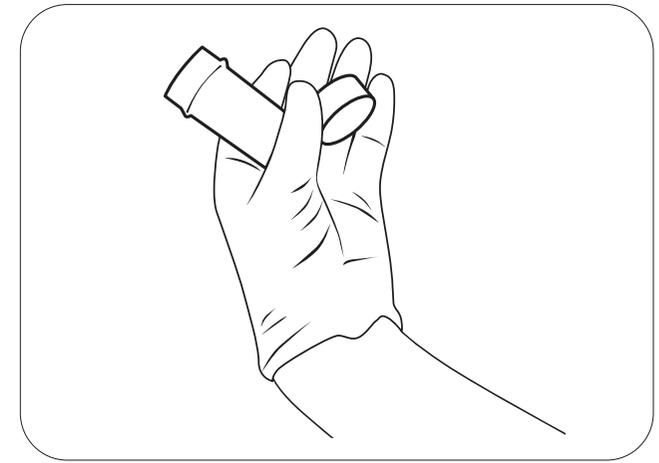


3. PREPARE

Prepare tube. Remove lid, taking care not to touch inside of the
lid or tube. Hold lid between thumb and finger

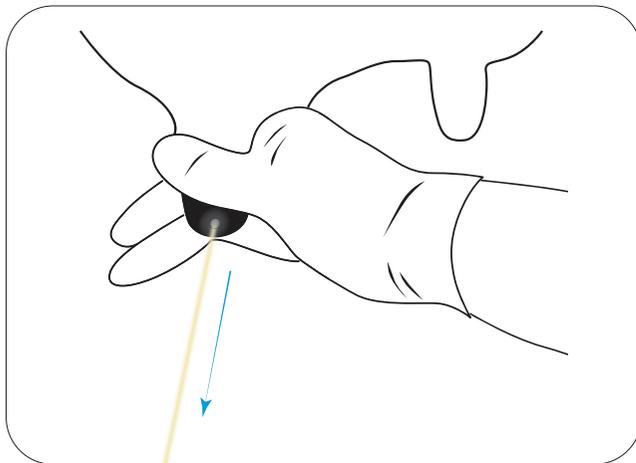


4. DISCARD

Discard the first 2 to 3 squirts of milk

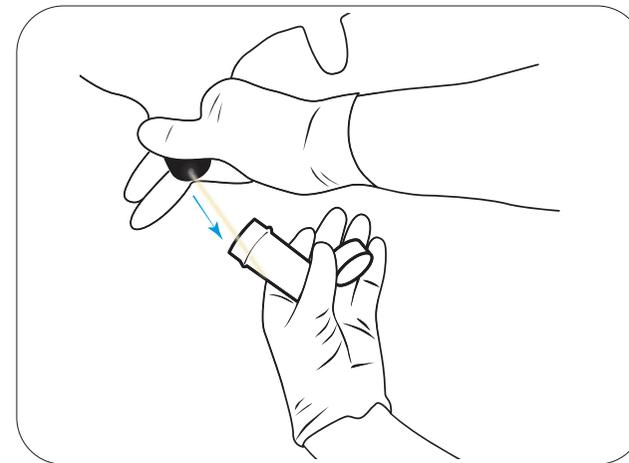

5. COLLECT

Hold pottle at 45 degree angle and half fill pottle with milk. Affix lid

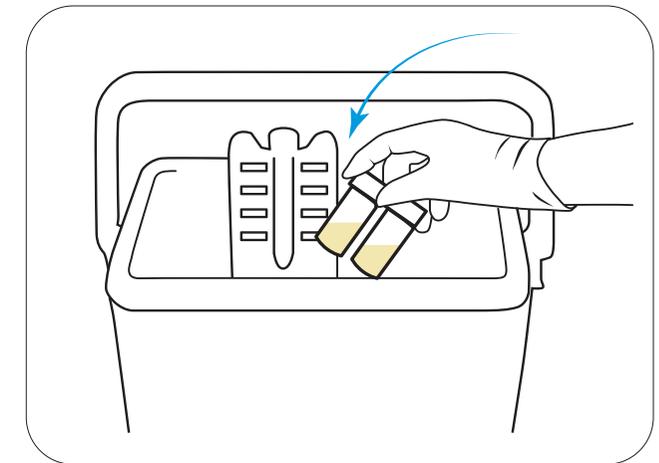

6. START TEST OR REFRIGERATE

Start Mastatest the same day, or alternatively refrigerate sample
and test later