



## HOW TO RUN MASTATEST

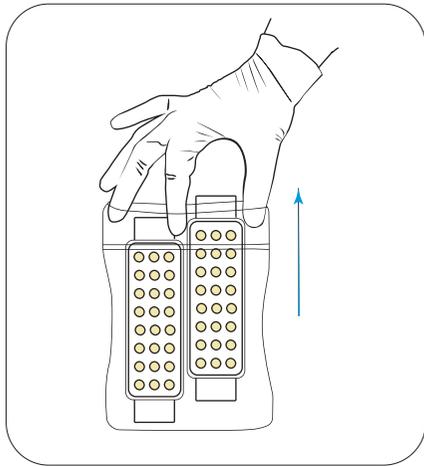


### 1. OPEN

Wearing clean gloves, remove cartridge from bag

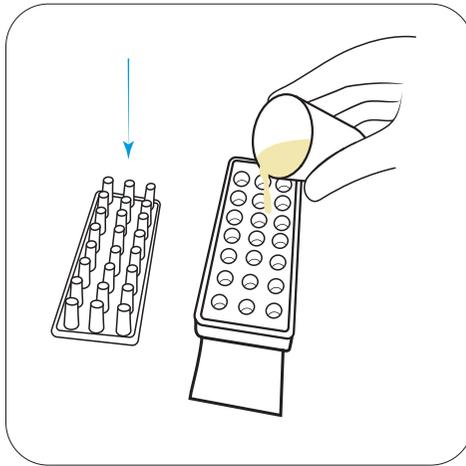


### 2. ADD MILK SAMPLE

Remove lid and place lid upside down (ensure you do not touch the plungers). Fill the mastatest with milk until half full

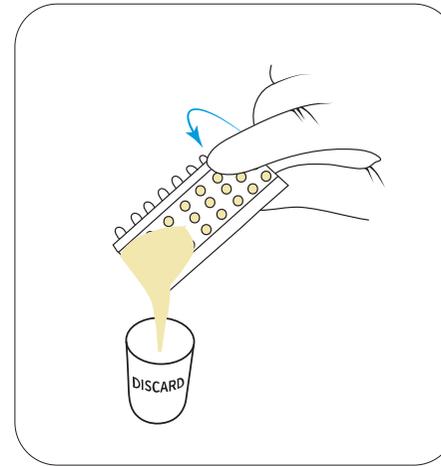


### 3. DISCARD SURPLUS MILK

Pour off surplus milk, and discard. Do not put surplus back into milk sample

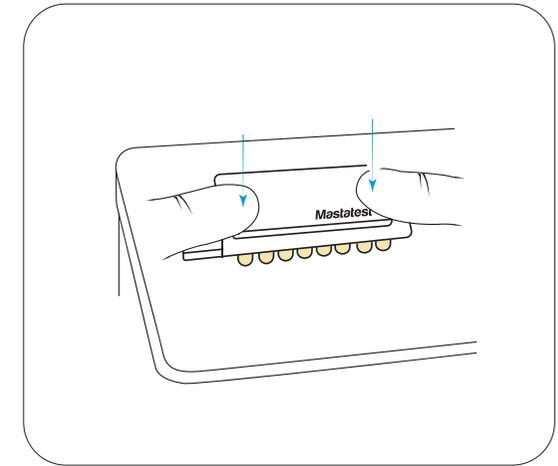


### 4. REPLACE LID

Replace lid (ensuring it is centered) and push down firmly onto bench

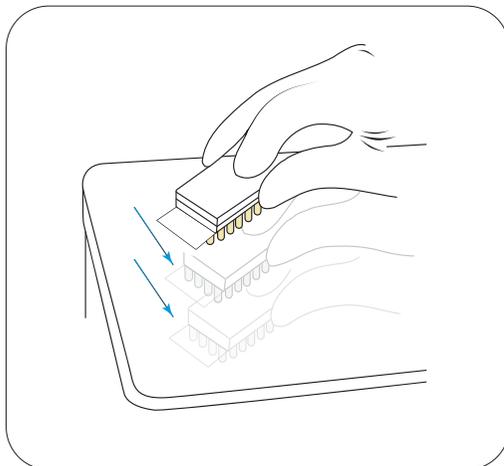

### 5. TAP FIRMLY

It is important to tap firmly, twice

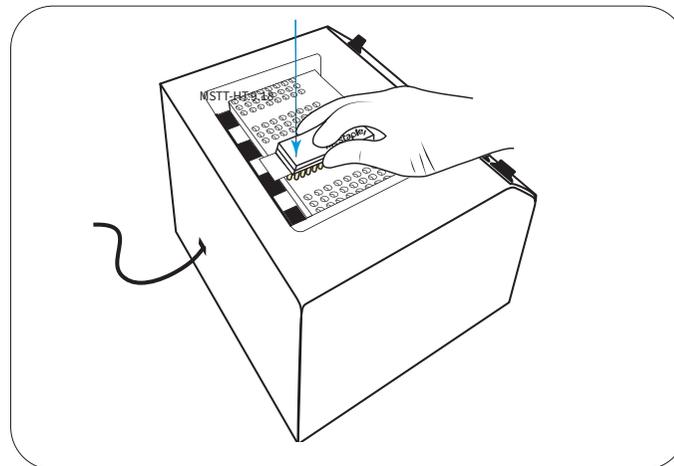

### 6. INSERT CARTRIDGE

Insert into Mastatest lapbox with QR code label facing down and toward the back of the lapbox



### 7. START

Dial in the cows number. Press enter to start test. Results sent via email within 24 hours