














Set-up Guide



1. Connect

- Connect the power cable, and ethernet cable (if available)

2. Set preferences

- Set operating language: scroll , then  to confirm
- If no ethernet is detected:
 -  to scan for Wifi networks, scroll  to select, then  to confirm
 - Input password: Use number keys, or scroll   to select characters. Move cursor using  
 -  to save password, then   to continue
- Note 16-digit Lapbox ID, then  to exit set-up

3. Register Lapbox

For new clients: Create your account at mastatestsystem.com

For existing customers:

- Login to your mastatestsystem.com account
- Select “My Account” and “Register a Lapbox” from menu





Caring for your Mastatest system


- Recommended room temperature of 5-30°C (41-86°F) for best performance
- Position Lapbox on a clean, dry desk space and handle with care
- Keep internal and external surfaces clean using a damp cloth (water only)
- Store cartridges at 2-8°C (35-46°F), and use before expiry date




For more information mastatest.com/user-guides or support@mastatest.com

User Information

-  Red light indicates power on.
-  Green light indicates internet connection.
-  to return to previous screen.
-  long-press to delete a character entry.

Set-up menu options are always available by pressing  when on main testing screen


Fill a Mastatest® cartridge

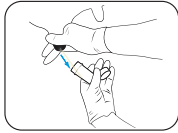


Take a clean milk sample
(see user videos)




Put on clean gloves, and remove
cartridge from pack

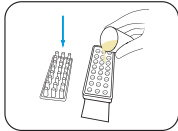


Remove cartridge lid and
half-fill cartridge with milk

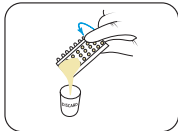

Pour off surplus milk and discard

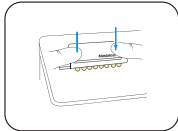

Replace lid, pushing down firmly,
and label lid with cow details

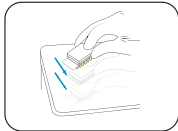

Tap firmly 2-3 times, repeating if
air-bubbles are visible in wells



Insert cartridge into Lapbox™
with arrows facing to the back
and replace lid

Start a Mastatest

1. Start position defaults to first available. **ENTER** to confirm,
or scroll **◀ ▶** to chosen position, then **ENTER**

Cartridge type should be automatically detected.

*If not select Single (1 test) **ENTER**
or Twin (2 tests) cartridge **▼ ENTER***

2. Select **BoxStart** **ENTER** , or **WebStart** **▼ ENTER**

3. Enter animal details:

- BoxStart: Enter animal number eg: **1 2 3** **ENTER**
and scroll to select quarter **▼ ENTER**
- WebStart: Enter at mastatestsystem.com

- When test has started, a countdown timer shows on screen. Further tests can be started at any time.
- **↶** to return to prior screen.
- Long-press **↶** to delete.
- To check or cancel a test in progress, scroll to select position **◀ ▶ ENTER**, then **ENTER** to check or **▼ ENTER** to cancel.

