




# How to fill a Mastatest® cartridge

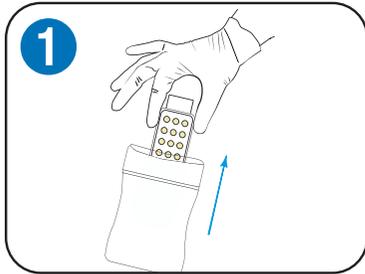


1 Put on clean gloves and remove cartridge from pack.

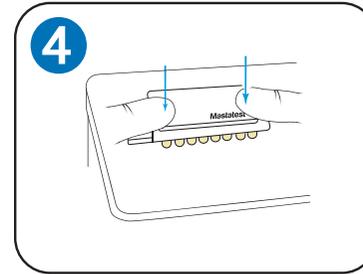


4 Press lid on firmly.  
Label with sample ID.

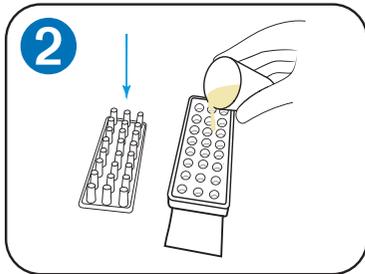

2 Remove cartridge lid, placing it on its flat back.  
Half-fill cartridge with milk.

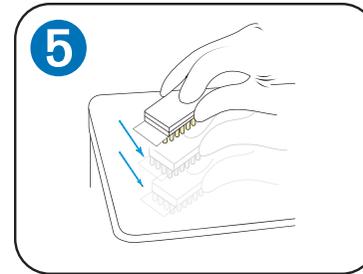

5 Tap firmly 2-3 times.

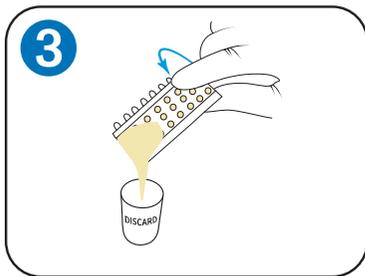

3 Pour off surplus milk and discard.



6 Place in Lapbox with arrows to the back, and replace lid.  
Start test.

Repeat steps 2 and 3 using the second milk sample, if you are starting a Twin (2 sample) test.



For further information, videos and user guides:  
[www.mastatest.com/customer-support](http://www.mastatest.com/customer-support)