

Set Up MastaCube®

Connection: WiFi ••)

1. Included in box



1 Power cable



1 Ethernet cable



1 MastaCube



2. Initial Set up



Connect your MastaCube to the power using the cable provided.



Optional:
Attach a USB
keyboard
(not included).



Turn power ON
at wall and wait for
MastaCube to initiate.

3. Set Language



On the **Language Settings** page, scroll to your preferred language using the grey buttons.

Hold (2s) the right button to select.

Press the right button to confirm the change, if prompted.

4. WiFi Setup

Select from one of these options to set up your 2.4GHz WiFi connection:

4A WiFi QR — Turn to page 2


4B Mobile — Turn to page 3

4C Keyboard — Turn to page 4

5. Register

Once connected to WiFi, register your MastaCube.
Turn to page 5.

4A: Set Up WiFi using a WiFi QR

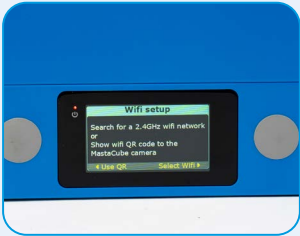

1. From the **WiFi Set Up** page, select **Use QR** by pressing the left grey button.



2. Remove the MastaCube cover by pressing the grey clips on either side, and lifting.



3. On your mobile phone, open the WiFi QR for any network for which you have the password stored.

On iPhone: Open the **Passwords** app
> Select **Wi-Fi** > Select the network
> Tap **Show Network QR Code**.

On Android*: Open the **Settings** app
> Select **Network Settings**
> Select **Network & internet**
(or **Connections**) > Select **Internet**
(or **WiFi**) > Select your network
> Tap **Share** the QR



4. Show the displayed QR code on your phone screen to the MastaCube camera.



5. Once the QR code has been scanned, the MastaCube screen will show it is **Connected**.

Select **OK** by pressing the right grey button.

Replace the MastaCube cover.

5. Register

Turn to page 5 for instructions on completing your registration.

*NB: Instructions may vary by phone manufacturer
- search for instructions specific to your model online

4B: Set Up WiFi using a Mobile Phone



1. From the **WiFi Set Up** page, press the right hand grey button to go to **Select WiFi**.



2. On the **Select Network** page, scroll to your preferred network using the grey buttons.
Hold (2s) the right button to select.



3. Using your mobile phone camera app scan the QR code that appears on the MastaCube.

Click the link on your phone to open it.



4. Enter your WiFi password, and click **generate QR code**



5. A QR code will appear on your phone screen



6. Remove the MastaCube cover by pressing the grey clips on either side, and lifting.



7. To **Scan QR**, show the generated QR code on your phone screen to the MastaCube camera.



8. Once the QR has scanned, the MastaCube screen will show it is **connected**.

Select **OK** by pressing the right grey button.

Replace the MastaCube cover.

5. Register

Turn to page 5 for instructions on completing your registration.

4C: Set Up WiFi using an attached Keyboard



1. Ensure you have your USB keyboard plugged into the MastaCube.



2. From the **WiFi Set Up** page, press the right hand grey button to go to **Select WiFi**.

Alternative: Use the right arrow on the keyboard to choose **Select WiFi**



3. On the **Select Network** page, scroll to your preferred network using the grey buttons.

Hold (2s) the right grey button to select.

Alternative: Use the arrow keys and ENTER on the keyboard to select.



4. On the **Scan QR** page, press the right grey button to select **Use Keyboard**.

Alternative: Press the right arrow on the keyboard to select **Use Keyboard**.



5. As prompted on the **WiFi Password** page, enter the network password using your keyboard, then press ENTER.



6. Once complete, the MastaCube will show it is **Configured**.
Select **OK** by pressing the right grey button.

Alternative: Press the right arrow on the keyboard to select **OK**.

5. Register

Turn to page 5 for instructions on completing your registration.

5: Register your MastaCube

Create your Mastatest account using one of these registration options:

5A: Mobile Phone QR




1. From the **Register** page, use your mobile phone camera to scan the QR code that appears on the MastaCube screen.

Click the link shown on the phone screen to be automatically directed to the registration page.



2. Complete the registration process. Once you have finished registering, press the right grey button to **continue** to the testing menu.

5B: Desktop or mobile browser

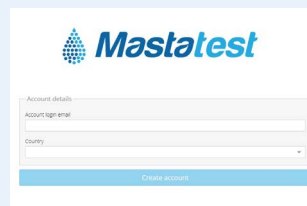

1. Open a web browser, and navigate to www.mastatest.com/register



2. Complete the registration process. Once you have finished registering, press the right grey button to **continue** to the testing menu.



- The **MastaCube ID** is displayed on the MastaCube screen.
- The **Serial Number** is on the bottom of the MastaCube.



If you have queries about the registration process, you can find a help guide [here](#).