


Set-up the Mastatest® Lapbox™ in 3 easy steps:


1. INSTALL LAPBOX

- Fold out the screen by gently pulling from underneath. (To fold away, push the silver buttons either side of Lapbox.)
- Connect Lapbox to ethernet (LAN) and power source using supplied cables (to connect via WiFi see overleaf)

2. REGISTER YOUR LAPBOX *(NB: if already completed, please skip this step)*

- Login, or create an account at mastatestsystem.com portal.
- From portal menu, select *My Account*, then *Register New Lapbox*. Then input:
 - Lapbox ID (from the Lapbox main menu, ▼ ▼ to select *Lapbox Information* )
 - Serial number (on rear of Lapbox)

3. CHECK CONNECTION

- Check green WiFi light is showing on the right hand side of Lapbox display. If not, check your connection.
- Test cloud connectivity (from the Lapbox main menu, scroll down ▼ ▼ to select *Test upload to Cloud*, then )

Lapbox is ready to use!

CARE

- Operate Lapbox within recommended room temperature range 5-30°C (41-86°F) for best performance
- Position your Lapbox on a clean, dry desk space and handle with care to protect camera inside
- Store Mastatest cartridges at 2-8°C (35-46°F), and use before expiry date
- Keep internal and external surfaces clean using a damp cloth (water only)



For more information mastatest.com/user-guides or support@mastatest.com











CONNECT VIA WiFi (NB: ethernet is recommended - see overleaf for connecting via ethernet)

Starting from the Lapbox main menu:

1. Configure for WiFi










- Scroll down    to *Configure Lapbox* 
- Select *Set internet connection* 
- Scroll down  to *Set WiFi*  and wait for Lapbox to restart

2. Set-up Network SSID

- Scroll down    to *Configure Lapbox*, then 
- Scroll down  to *Setup WiFi Connection*, then 
-  to scan for networks, then scroll  to your chosen network, and  to select
- Go back  to *Setup WiFi Connection*

NB: > -65dBm, and 2.4 GHz WiFi required

3. Enter network password

- Scroll down  to *Set WiFi password*, then 
- Input WiFi password using   and  to select characters. Long-press  when complete
- Press  again to save
- Go back   to main menu

4. Check connection

- Scroll up   to *Test upload to Cloud*, then 



For more information mastatest.com/user-guides or support@mastatest.com