



Fill a Mastatest® cartridge

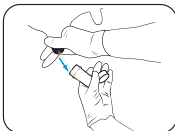


Take a clean milk sample
(see user videos)




Put on clean gloves, and remove
cartridge from pack

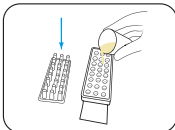


Remove cartridge lid and
half-fill cartridge with milk

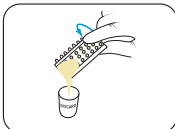

Pour off surplus milk and discard

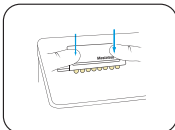

Replace lid, pushing down firmly,
and label lid with cow details

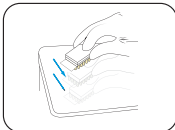

Tap firmly 2-3 times, repeating if
air-bubbles are visible in wells



Insert cartridge into Lapbox™
with arrows facing to the back
and replace lid

Start a Mastatest®

To Begin:

- From the Lapbox main menu, select **Start Mastatest**
- Select Lapbox slot used: If first free slot used ,
or scroll to select and

Option 1: BoxStart

- Select test type:
 - For Clinical
 - For HiSCC
- Select **BoxStart**
- Input cow number (e.g. 00123)
- Scroll down to select quarter
- Confirm start

Option 2: QuickStart

- Scroll down to select test type:
 - For Clinical **QuickCM**
 - For HiSCC **QuickSCC**
- Input test details to mastatestsystem.com within 21 hours

Next steps:

- Countdown timer shows the remaining time
- Further tests can be started independently
- Results emailed within 24 hours



Further information and support

Mastatest.com/user-guides

support@mastatest.com